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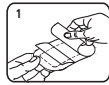
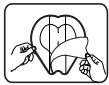
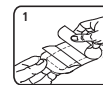
EN

PolyMem® Silver Film Adhesive Dressing

Step 1. Remove one side of adhesive release liner.

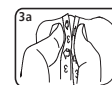
Sacral/Oval #8: Remove the center liner. Note the dressing is perforated to allow folding.


Sacral/Oval #8: Gently fold the dressing in half and hold the dressing by the two "wings".


Step 2. Apply over wound. Slowly remove remaining release liner(s) while pressing adhesive to the skin.

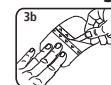
Step 3a. Pinch dressing slightly where two white strips come together at edge of dressing. This will cause cover sheet to start to lift.

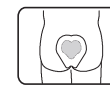Step 3b. Remove cover sheets one at a time. Gently smooth thin film as each cover sheet is being removed.

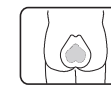NOTE: Sacral dressing is designed to be applied in different orientations to best cover the wound. The dressing may also be modified or trimmed to accommodate different wound shapes.

DESCRIPTION

PolyMem polymeric membrane dressings are made of moisture-loving polyurethane matrix that contains (1) safe wound cleanser, (2) glycerin moisturizer, and (3) superabsorbents. Dressings have a thin polyurethane backing which breathes and helps keep out contamination. Dressings may be used as a primary, secondary, or combined primary and secondary dressing. PolyMem MAX™ dressings have the same polymeric membrane as PolyMem dressings, but MAX is thicker and has higher MVTR to handle more fluid.

PolyMem Silver products incorporate very fine silver particles, and will not stain skin. Silver acts inside the dressing to protect from microbial contamination. Approximate silver content by area (µg/cm²) PolyMem Silver 124 PolyMem Silver MAX 207. Drug-free PolyMem dressings help to:

- effectively manage and heal wounds
- absorb fluid and provide a moist healing environment
- minimize and relieve persistent and procedural wound pain
- reduce edema, bruising, and the spread of inflammation into surrounding undamaged tissues

INDICATIONS FOR ALL HUMAN BEINGS:

Under the direction of a healthcare professional, for the management of serious wounds, such as most partial and full thickness wounds, including:

- Pressure ulcers (stages I-IV) • Venous stasis ulcers • Acute wounds
- Leg ulcers • Donor and graft sites • Skin tears • Diabetic ulcers
- Dermatologic disorders • 1st and 2nd degree burns
- Surgical wounds • Vascular ulcers

PRECAUTIONS

- For external use only • PolyMem dressings are not compatible with oxidizing agents (hydrogen peroxide or hypochlorite solutions)

- Prior to performing imaging procedures, determine if the dressing should be removed • Avoid contact with electrodes or conductive gels
- Topical treatments are not recommended in conjunction with PolyMem dressings • Be alert for signs of infection or maceration • Do not use and discontinue use on people who show signs of sensitivity, irritation, or allergy from the dressing or its materials • Intended for one use on one person. Do not reuse. Reuse may degrade properties, spread contamination, and increase risk of infection

PolyMem dressings may be used when signs of infection are present if proper medical treatment that addresses the cause of the infection has been implemented.

PolyMem Silver dressings are recommended when a wound displays delayed healing, signs of infection, or increased bioburden; or the patient has infection risk factors (such as contaminated wound, poor nutrition, diabetes, history of infection, etc.).

Store at room temperature ≤30°C (86°F).

Excursions permitted ≤55°C (131°F) but should be minimized.

Dispose as typical for biohazardous waste.

Dimensions are approximate.

DIRECTIONS

Initial Application

1. Prepare the wound according to protocol or as directed by a physician or other ordering clinician.
2. Select a dressing with the membrane pad ¼" – 2" (0.6 – 5 cm) larger than the wound. The dressing should also cover any inflamed or damaged area surrounding the wound.
3. For dry wounds, moisten dressing slightly or leave wound moist after preparation. For good adhesion, ensure skin is clean and dry.

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PolyMem is designed to continuously clean the wound and does not leave residue that needs to be removed. **Excessive cleaning may injure regenerating tissue and delay wound healing.**

Observations during use or change

- During normal healing, the wound may appear to become larger during the first few dressing changes because the body is naturally removing non-viable tissue. If the wound continues to become larger, consult an expert.
- PolyMem dressings help support new blood vessel formation so it is not uncommon to see blood-stained wound fluid and dressings during changes.
- Be alert for signs and symptoms, such as, increased redness, inflammation, pain, warmth, odor, whitening, fragility, excessive tissue formation, or lack of healing. Consult a professional for problem situations, such as, infection, irritation, sensitivity, allergy, maceration, or hypergranulation.

Any serious incident that has occurred in relation to this device should be reported, by the user and/or patient, to Ferris Mfg. Corp. and the competent authority of the Member State in which the user and/or patient is established.